




## User Manual

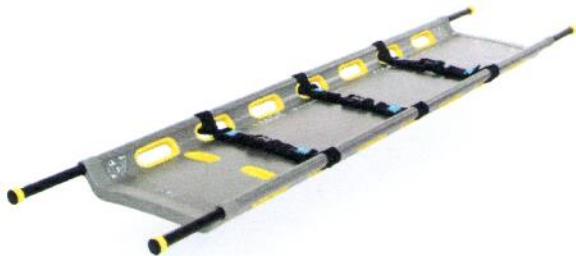

### LS 4001

#### Stretcher with retractable handles and safety straps (art. 4012)

The LESSO stretcher is used for manual patient evacuation in close consultation with medical personnel. Operators should practice regularly in use of this equipment according to company/ organization rescue routines. The LESSO stretcher is designed for the use of minimum 2 operators. Patient need to be secured with safety straps before lifting. The LESSO carrying harness (3013) is an approved carrying accessory.

**Wrong use of any patient handling equipment can cause severe injury hence special care and attention should apply.**

The LESSO stretcher is ideally stored indoor, with handles inside, and in a dark and dry environment between -25C to +35C.

After its use, the LESSO stretcher should be cleaned and checked before returned to its usual storage, to restore preparedness.

Any damage to the stretcher to be reported to the HES officer without delay.

150kg

Bra løftekapasitet



Røntgenpenetrerbar



Lett å rengjøre



Varmeisolerende



20kg

Oppdrift



Lett å bære



Resirkulerbar

The LESSO stretcher has 2 holes in shoulder position for vehicle safety belt to run through and ensure that patient is secured during transportation.



Handles are retractable with a built-in stopper. Handles are locked in position by turning the handle either way.

Handles should **always** be locked in position before carrying a patient.

The LESSO stretcher can still be used during CT-scanning and X-ray, but not in MR-examination due to materials in stretcher frame.

#### Manufacturer

LESS AS

P.O.Box 113, NO-2858 Kapp

NORWAY

Phone: +47 611 60 055

[www.less.no](http://www.less.no)

[post@less.no](mailto:post@less.no)