Electronic Instruction For Use

* Please note that the images in this online instruction for use (eifu) are for illustrative purposes only and may differ slightly from the actual product you receive. While we strive to ensure the accuracy and completeness of the information provided, we cannot guarantee that the image accurately reflects the product at all times.

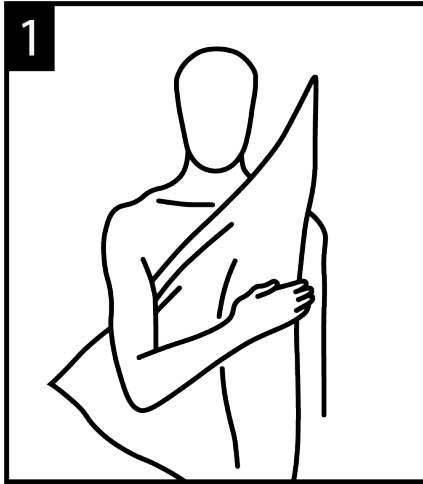
However, we want to assure you that through a continuous risk assessment process, we have evaluated that the potential deviations between the images and the actual product do not pose an increased risk when using the product. The instructions and essential information provided in the eifu remain valid and applicable, ensuring the safe and effective use of the product.

NORSE RESCUE® Triangular Bandage


 English	2	 Norsk (Norwegian)	6	 Hrvatski (Croatian)	11
 Français (French)	2	 Suomi (Finnish)	7	 Eesti (Estonian)	11
 Español (Spanish)	3	 Ελληνικά (Greek)	7	 Latviešu (Latvian)	12
 Italiano (Italian)	3	 Български (Bulgarian)	8	 Lietuvių (Lithuanian)	12
 Português (Portuguese)	4	 Čeština (Czech)	8	 Slovenščina (Slovenian)	13
 Nederlands (Dutch)	4	 Slovenčina (Slovak)	9	 Malti (Maltese)	13
 Deutsch (German)	5	 Magyar (Hungarian)	9	 Gaeilge (Irish)	14
 Svenska (Swedish)	5	 Polski (Polish)	10	 Íslenska (Icelandic)	14
 Dansk (Danish)	6	 Română (Romanian)	10		


Instruction For Use

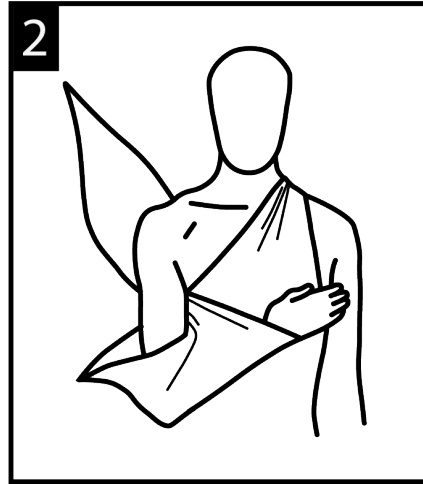1 Support the patients arm across the patients chest. Put the bandage under the patients arm and around the back of the patients neck.

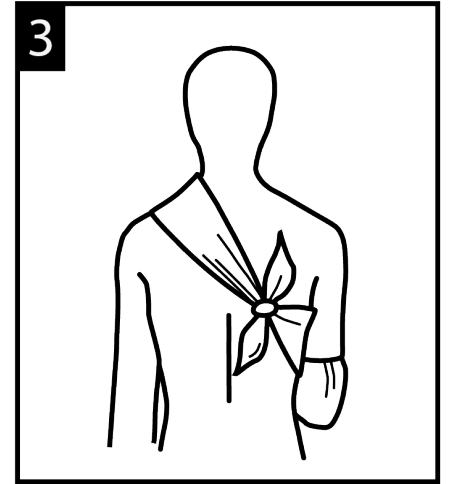2 Put the other half of the bandage over the injured arm and through the armpit to be tied on the back.


3 Tie a knot on the lower part of the patients back to increase support.

Tuck the loose ends of the bandage in at the elbow or use a pin.

ATTENTION:

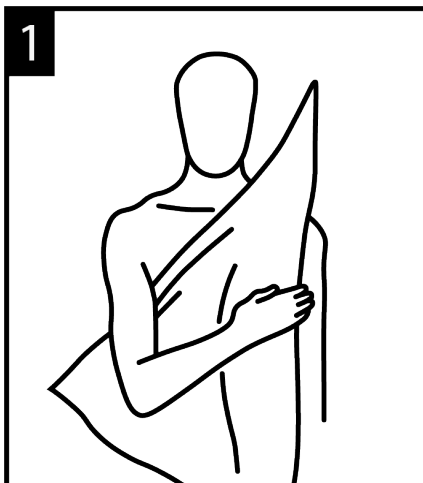If you are using a triangular bandage to support a lower limb or large dressing, fold it in half horizontally so the point of the triangle touches the middle of the long edge. Then fold it in half again in the same direction to make a broad strip.

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AP Services A/S